

### Applications

Exposed pulp in pulp capping and pulpotomy leakage canal apexification formation of hard tissue barrier root canal filling material

### Directions for use

Clean and dry it, after preparing the root canal Set up a disposable tip in syringe and Disinfect the disposable tip with ethanol Fill the canal completely by pushing the syringe plunger while withdrawing the tip slowly Clean any excess paste with a sterilized cotton pellet Remove the used tip, fix a new one and cover up If contact with air, as content could be hardened, so it is necessary to cover a cap on the needle after use.

### Advantage

Easy cleaning and removal

Excellent antibacterial effect and radiopacity Premixed paste in a convenient syringe

Excellent accessibility to the root canal and prevention of cross-contamination

### Contents

2 x 1.5g paste in syringe

Disposable tips

### Storage

Do not fill beyond the apex

### For dental use only

Do not use in allergic person keep the syringe capped at all times, when not in use Store in a dry and cool area to avoid dryness or separation of the paste

### Manufactured by



**RIDENT DENPRO PVT. LTD.**



40-41, Vrindavan Nagar, B/H Jain Temple,  
Boranada Industrial Area, Jodhpur-342008  
Rajasthan, INDIA.

[www.rident.in](http://www.rident.in)